

**TOUCHSTONE™**

**APS**



# ***Auto Purse String Clamp***

Code: APS45 / APS65

# Auto Purse String Clamp

## Unique Features

- Just clamp. Pre-loaded with suture and staples
- Avoid contamination
- Reduce OR Time
- Eliminates manual purse-string
- A neat & concise pure string everytime

## Application and Principle

- For application in all facets of gastro-intestinal surgery and wherever a purse-string placement is required.
- The clamp places suture around the intended tissue site by small staples
- Open the instrument jaws by spreading the ring handles, remove the foam buffer, and place the instrument jaws across the intended tissue site and position appropriately.
- Squeeze the ring handles until they lock with a 'click' sound. Staples are attached to the tissue site holding the suture in place.
- Release the purse-string clamp form the site. The stapled suture is now ready for tying down.





APS

TOUCHSTONE™



## APS

Order Code	Handle Length	Cartridge Length	Suture Length	Staple Quantity
 <p>APS45</p>	310mm	47mm	690mm	18
 <p>APS65</p>	256mm	67mm	730mm	28

### TOUCHSTONE™

Touchstone International Medical Science Co., Ltd.

21A Science Plaza, International Science Park,  
No.1355 Jinjihu Avenue, Suzhou, 215021, P.R. China  
Tel: +86 512 6299 1933; 6299 1939



CE 0123